

CODE: WC1220  
ISSUED: 2009 NOV  
ABLE RATING **AMBER**  
NON DANGEROUS GOODS



★ ALTERNATIVES OR SIMILAR ★  
★ WESTGUARD                      ★ COMET  
★ BIO-DINE                         ★ Q-SAN

## INFORMATION SHEET

# VETO

## HOSPITAL AND LABORATORY DISINFECTANT

### DESCRIPTION

**VETO** is a glutaraldehyde-based sanitizer specially developed for disinfecting instruments in hospitals, medical and dental laboratories, etc. Also ideal for disinfecting pounds, vets, kennels.

**VETO** has been tested by the Department of Agriculture of Western Australia and proven to inactivate canine parvovirus at the recommended dilution rates. **VETO** can also be used against feline panleukopaenia and porcine parvovirus.

**VETO** contains 2% glutaraldehyde for fast and effective sanitisation and is non-corrosive to all materials used in medical and dental instruments.

### RECOMMENDED USE

It is recommended that **VETO** be used neat for disinfecting instruments. Pre-clean instruments, then rinse with water. Immerse instruments in **VETO** and leave for at least 10 minutes. Remove instruments and rinse well with clean water. **VETO** solutions may be re-used for up to one week unless heavily contaminated. Replace solutions regularly.

As kennel wash: use at a dilution of 1:10 in clean water. Spray or swab onto the surface and leave for 20 minutes contact time. Rinse off with clean water. Discard made-up solutions after use.

### SAFETY DIRECTIONS


WARNING - Irritant. Gloves and protective goggles must be worn when handling **VETO**. Avoid contact with skin and eyes. Avoid inhaling vapour or spray mist. Ensure adequate ventilation when using. **DO NOT SWALLOW.**

### FIRST AID

If poisoning occurs, contact a doctor or Poisons Information Centre. If swallowed, do NOT induce vomiting. Give water or milk, then raw egg.

### PACKAGING

**VETO** is available in 5 and 25 litre containers.

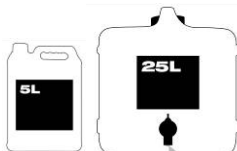

### BENEFITS & TIPS

- DEPT AGRICULTURE DEVELOPED & APPROVED
- KILLS PARVO VIRUS
- MUST HAVE FOR KENNELS & POUNDS
- CONCENTRATED
- PROVEN FORMULA



PROUDLY AUSTRALIAN OWNED & MANUFACTURED BY ABLE WESTCHEM  
ABN 82 009 353 182. 47 BANKSIA RD, WELSHPOOL WA 6106  
Ph 08 9458 1604 Fax 08 9458 1222 Email: [info@westchem.com.au](mailto:info@westchem.com.au)  
Web 1: [www.westchem.com.au](http://www.westchem.com.au) Web 2: [www.envirocareearth.com.au](http://www.envirocareearth.com.au)

**ALWAYS READ MSDS**

Page 1 of 1

